



Veltop Modulo 3

Assembly instructions

Thank you for choosing the Veltop Modulo 3 model.

This product is designed, manufactured, and assembled by CYCLO-SYSTEM, located in Haute-Savoie (74), France. To ensure proper use, optimal performance, and a long life for your new accessory, please read and carefully follow the usage and assembly instructions.

This model can be installed on any mobility scooters compatible with the front and rear mounts listed below.



Assembly video

Warning

Failure to follow this manual may impair the protection's functionality and cause certain parts to fail. This could compromise the effectiveness of the system and endanger your safety during travel.

Choice of two rear mounts



Seat square tube fixation




Headrest fixation

Front mounting



Plate fixation

Product contents

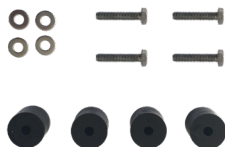

Screws / Spacers / Washers



Windshield



Allen key




Side panels with armrest fixations



Front structur



Tubular structure

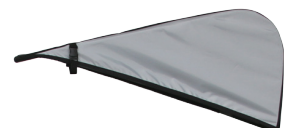

Canopy

1

Mounting rear attachments

Rear attachment to the headrest

Remove the scooter's headrest, insert the tubes between the two mounting plates, and tighten. Then, place the headrest back into position.



Rear attachment on the square tube under the seat

Place the square tube welded to the plate under the seat so that the vertical tube can be centred in the middle of the seat, then screw in place.



2

Mounting front attachment

Remove the basket and its support.

Place the plate using two or four screws, inserting spacers between the plate and the scooter console.



3

Mounting the front structure and windscreen

Place the lower structure into the top of the two brackets and tighten.

Insert the upper windscreen into the lower structure and tighten the two adjustment knobs.



4

Mounting the tubular structure

Assemble the tubular structure and centre it in the brackets of the rear attachment.



5

Installing the canopy

Attach the canopy to the frame, starting from the front.

Then, place the elastic bands on the horizontal tube.



6

Attaching the side panels

Use the zipper to attach the side protections to the canopy.

Place the provided straps on your seat's armrest. Insert the strap into the buckle to secure the side panels.



bbbb



Adjustments

You can adjust the height of the windshield and canopy. Leave a space of approximately 2 inches between the canopy and the windshield to prevent contact when turning the handlebar.

In case of poor visibility (snow, frost, ...), lower the windshield to clear your field of vision.



Possible configurations



Optional basket mount support

Place the rounded part of the support into the lower part of the front plate brackets, and then tighten.

Reattach your basket support to the top of the front plate before placing your basket.



CYCLO-SYSTEM

3621 route d'Albertville
74320 SEVRIER
FRANCE

T : 0033 (0)4 50 77 56 17

F : 0033 (0)4 50 63 45 36

info@veltop.com

www.veltop.eu/en



SKU: VELMOD3