



Veltop Comfort Mobil

Assembly instructions



Thank you for choosing the Veltop Comfort Mobil model.

This product is designed, manufactured and assembled by CYCLO-SYSTEM, located in France. To ensure proper use and operation and to guarantee a long lifespan for your new accessory, please carefully read and follow the instructions for use and assembly.

Please note that this protection system is designed to protect the user from rain, cold and sun. This model can be installed on all mobility scooters equipped with a removable front basket and either a headrest or a square tube under the seat at the rear.

Avertissement

Failure to follow these instructions may impair the proper functioning of the protection system and cause certain parts to fail. This could reduce the system's effectiveness and compromise your safety while travelling.

Two fixing options available:



Headrest attachment



Square tube attachment

Comfort Mobil - A



Canopy + Cap



10 mm spanner



Canopy support tube



2 Allen keys



Canopy



10 mm spanner



Canopy support tube



2 Allen keys



Side panels



Transparent front protection



Canopy



10 mm spanner



Canopy support tube



2 Allen keys



Side panels



Transparent front protection



Polycarbonate front shield



Screws

1 Installing of the rear attachments available

Installing the headrest fixing

Remove the headrest, insert the two tubes between the two fixing plates, then replace the headrest.

Center the fixing and tighten the 3 screws using the Allen key and a 10 mm spanner.



Installing the fixing on the square tube under the seat

Insert the square tube welded to the plate into the tube located under the seat.

Tighten the clamping screw of the scooter's square tube to secure the assembly.
Position the vertical tube centrally and fasten it with the 2 screws.



2 Installing the canopy support tube

Remove the two half-shell clamps from the lower fixing parts, then position and horizontally center the tube.

Replace the two half-shell clamps and tighten firmly.



3 Installing the canopy

Position and insert the canopy plates into the two supports at the end of the tube, then tighten the two tightening knobs.



4**Installing the transparent front protection (configuration B)**

Attach the front protection to the canopy using the zipper, then install the two side panels. At the rear of the canopy, pass the strap under the tube and clip it to secure the canopy.

**5****Installing the front shield (configuration C)**

Remove the basket support. Position the front shield as high as possible, approximately 2 feet from the transparent front protection. If necessary, place the supplied spacers between the scooter console and the polycarbonate shield.

Reinstall the basket support in front of the shield and tighten using the scooter's 2 or 4 screws or the supplied screws.




Adjustments

You can adjust the canopy height by modifying the position of:

- the fixing parts located on the plate attached to the headrest
- the plate fixed to the vertical square tube

You can also slightly adjust the angle of the canopy support tube by loosening the rear fixing parts. The canopy should be positioned approximately 2 feet above the head to ensure optimal protection.

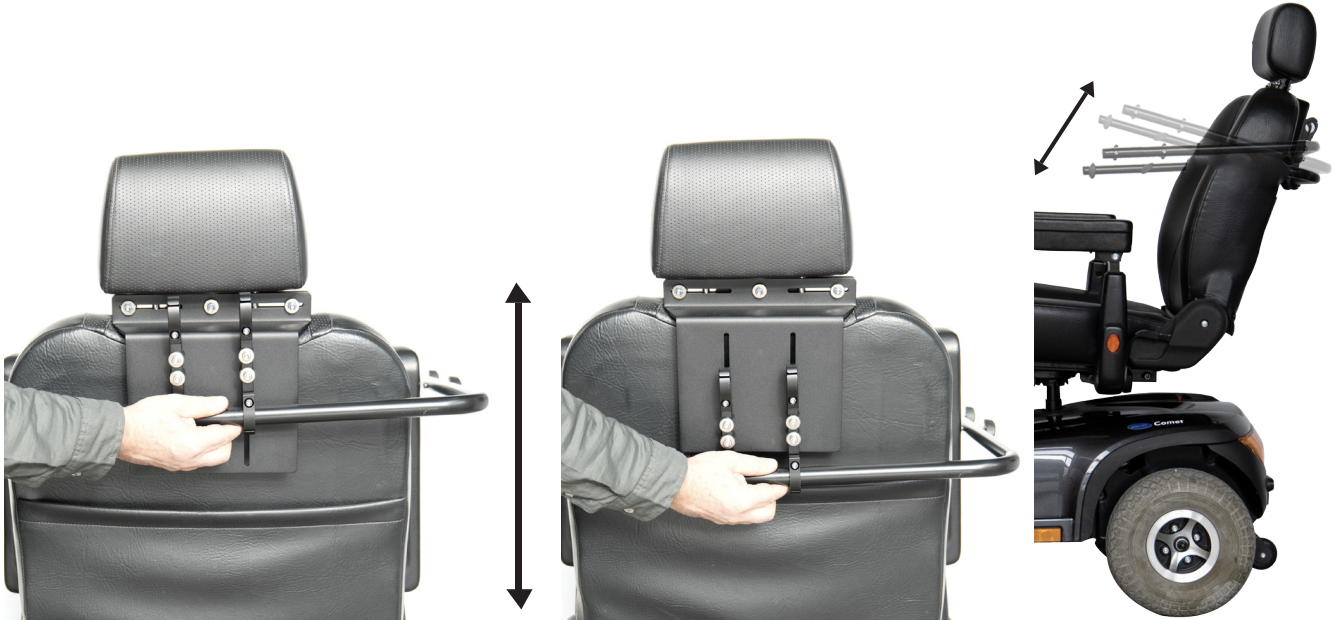

Configurations



CYCLO-SYSTEM

3621 route d'Albertville
74320 SEVRIER
FRANCE

T : 0033 (0)4 50 77 56 17

F : 0033 (0)4 50 63 45 36

info@veltop.com

www.veltop.eu/en


VELTOP^{eu}
WEATHER PROTECTION



CE

Ref. VELCM